



Colours will vary as pictured in this manual

# Premium 3-Fold Portable Massage Table

## User Manual

RETAIN THIS MANUAL FOR FUTURE REFERENCE

PLEASE READ THIS MANUAL CAREFULLY BEFORE USE

# Table of Contents

Safety..... 1

Parts Diagram ..... 1

Setting-Up Your Table ..... 2

Folding and Transporting the Table..... 3

Specifications ..... 4

# Safety

- Maintain the table periodically. Make sure all screws are secure and the cable has not deteriorated or frayed. Make sure there are no cracks on the aluminium structure.
- Do not overload the table beyond the recommended weight limit. This could result in serious injury to the user. This product is tested to hold a max working load of 250kg.
- Do not use alcohol based cleaners when cleaning the vinyl. This could cause damage to the vinyl material. Use warm soapy water and wipe dry.
- Make sure when using the table that all four legs are at the same level. Using the legs at different levels can cause the table to be unstable and cause injury.
- Do not allow children to assemble or disassemble the massage table for you unless under supervision.
- Be careful when using sharp objects as they can pierce the covering. Keep the table's covering away from being contacted by the sharp items.

# Parts Diagram



- |                     |                      |
|---------------------|----------------------|
| 1. Handle           | 5. Headrest          |
| 2. Copper button    | 6. Cable             |
| 3. Hinge            | 7. Side arm supports |
| 4. Hanged arm sling |                      |

# Setting-Up Your Table

Steps are as below:



1. Release the buckle.
2. Lay the table on its side with four rubber feet on the floor, open up the table with four legs extended as full as possible.
3. Take out all accessories and then lift the table up until all the legs are on the floor.
4. Put the headrest cradle into the corresponding outlets, position the cradle at an ideal angle for the face to sit comfortably. Lock in position by pressing the lever mechanism down (see figure below). Then put the headrest cushion on the cradle.



5. Remove the male clip attachments from the straps on the hanging arm rest. Re-attach them by inserting the strap through the underside of the clip's front slot, and down through the upper side of the back slot (see figure below). Then attach the hanged armrest or sling onto the headrest cradle by looping the straps through the bar and connecting the clips.



6. Insert the two side arm supports into the two holes at each side of the table.
7. The table is now ready for use.

The height of table is adjustable by pressing the copper buttons on the legs. All four legs should be done in the same way one by one and make sure all legs are adjusted to the same height level.

## Folding and Transporting the Table



1. Take out all accessories.
2. Lay the table on its side with four rubber feet on the ground.
3. Put all accessories under the elastic straps under the top veneer.
4. Fold the table up and lock the buckle.
5. Put the massage table into the carry bag.
6. Carry the table like a suitcase by grasping the handle.

**WARNING!** If the frame/legs/table structure is damaged in anyway, then you should not use this table. Using this table when damaged may cause harm or serious injury to you or your patient. You should seek repair immediately.



# Specifications

Material

3 inch high quality multi-density foam

100% polyurethane vinyl

Working Weight  
Capacity

250kg / 500lb



**Some experts believe that the incorrect or prolonged use of almost any product may cause serious injury or death. To help reduce your risk of serious injury or death, refer to the information below. For more information, see [www.datastreamserver.com/safety](http://www.datastreamserver.com/safety)**

- Consult all documentation, packaging and product labelling before use. Note that some products feature documentation available online. It is recommended to print and retain the documentation.
- Before each use, check the product for loose/broken/damaged/missing parts, wear or leaks (if applicable). Never use a product with loose/broken/damaged/missing parts, wear or leaks.
- Products must be inspected and serviced (if applicable) by a qualified technician every 6 months. This is based on average residential use by persons of average size and strength, and on a property of average metropolitan size. Use beyond these recommendations may require more frequent inspections/servicing.
- Ensure that all users of the product have completed a suitable industry recognised training course before being allowed access to the product.
- The product has been supplied by a general merchandise retailer that may not be familiar with your specific application or description of application. Be sure to attain third-party approval from a qualified specialist for your application before use, regardless of any assurances from the retailer or its representatives.
- This product is not intended for use where fail-safe operation is required. As with any product (for example, automobile, computer, toaster), there is the possibility of technical issues that may require the repair or replacement of parts, or the product itself. If the possibility of such failure and the associated time it may take to rectify could in any way inconvenience the user, business or employee, or financially affect the user, business or employee, then the product is not suitable for your requirements. This product is not intended for use where incorrect operation or a failure of any kind, including but not limited to, a condition requiring product return, replacement, parts replacement or service by a technician may cause financial loss, loss of employee time or an inconvenience requiring compensation.
- If this product has been purchased in error when considering the information presented here, contact the retailer directly for details of their returns policy, if required.



©2018 HPF. All rights reserved. No part of this document, including descriptive content, concepts, ideas, diagrams or images may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying, scanning or recording, or any information storage and retrieval system, without express permission or consent from the publisher.